

Sharing Blood Glucose Reports with your Healthcare Professional

With the CONTOUR®NEXT ONE and CONTOUR®DIABETES app and meter system, you have the ability to send your results to your Healthcare Professional in advance or share it on the day of your appointment.

1 From My Readings, tap the **Main Menu** icon.

2 From the Main Menu, tap on **My Care**.

3 Then tap on **Reports**.

4 Select: **Blood Sugar Diary (PDF)**
Your Healthcare Professional may also ask you to provide one of the other reports available.

5 Tap on **Full name of patient** and type your name.
Select the timeframe recommended by your Healthcare Professional and tap **View Diary**.

6 The report will be displayed on your mobile device. To share it with your Healthcare Professional, tap the **Share** icon.

Please e-mail it to:



If you require assistance, please contact the Ascensia Diabetes Care Customer Support Team at 1-800-268-7200 from 9:00 a.m. to 9:00 p.m. EST, Monday to Saturday.

To learn more about your CONTOUR®NEXT ONE meter and the CONTOUR®DIABETES app, go to contournextone.ca

© TM see www.ascensiadiabetes.ca/tm-mc

