**Contour next ONE**  
Blood Glucose Monitoring System

Works with the CONTOUR™ DIABETES app available for download on the App Store™ or on Google Play™.

Uses only CONTOUR®NEXT blood glucose test strips.

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**Your CONTOUR®NEXT ONE meter**

**Sample Tip:** Blood sample pulled in here

**CAUTION**  
Your CONTOUR®NEXT ONE meter works only with CONTOUR®NEXT test strips and CONTOUR®NEXT control solution.

Read your CONTOUR NEXT ONE User Guide for the *Intended Use* of the meter and a list of *Important Safety Information*, Warnings, and Precautions.

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**The CONTOUR™ DIABETES app**

You can automatically transfer results from your CONTOUR NEXT ONE meter wirelessly to your smartphone or tablet.

- On your smartphone or tablet, go to the App Store™ or the Google Play™ store.
- Search for the CONTOUR DIABETES app.
- Install the CONTOUR DIABETES app and follow the instructions to pair your meter.

OR Scan with QR Code app or visit [www.contourone.com](http://www.contourone.com) to learn more.

**NOTE:** If you do not pair your meter with the CONTOUR DIABETES app, follow the instructions in *Initial Setup from the Meter* the first time you turn on your meter.

**Turn the Meter On and Off**

- Press and hold the **OK button** for 2 seconds until your meter turns **On** or **Off**.
- Insert a test strip into the meter to turn the meter **On**.
- Press **OK** twice when the meter is **Off** to turn **On** the white test strip port light.

**Initial Setup from the Meter**

Press and hold the **OK button** for 2 seconds until the meter turns on.

**NOTE:** The screen dims after 30 seconds of inactivity. To light the screen again, press the **OK button**.

**Initial setup begins with View Pre-Set Target.**

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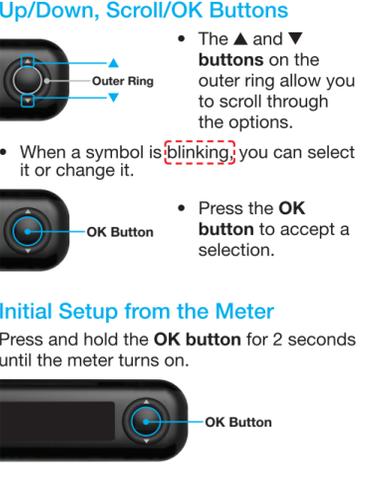
**Up/Down, Scroll/OK Buttons**

- The **▲** and **▼** buttons on the outer ring allow you to scroll through the options.
- When a symbol is **blinking**, you can select it or change it.
- Press the **OK button** to accept a selection.

**Initial Setup from the Meter**

Press and hold the **OK button** for 2 seconds until the meter turns on.

**NOTE:** The screen dims after 30 seconds of inactivity. To light the screen again, press the **OK button**.

**Initial setup begins with View Pre-Set Target.**

**View Pre-Set Target**

The meter compares your blood sugar results with a **Target Range** (for example, 3.9 mmol/L–7.2 mmol/L) to let you know if your test result is:

- within**  your **Target Range**.
- above**  your **Target Range**.
- below**  your **Target Range**.

You can change this pre-set **Target Range** from the CONTOUR DIABETES app after initial setup of the meter.

The meter displays a pre-set **Overall Target Range**.

- To move to the next screen, press the **OK button**.

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**Set the Time**

The hour is **blinking**.

- To change the hour, press the **▲** or **▼ button** on the outer ring.
- To set the hour and move to minutes, press the **OK button**.
- To change the minutes, press the **▲** or **▼ button** on the outer ring, then press the **OK button**.

For a 12-hour format, select **AM** or **PM**, press the **▲** or **▼ button** on the outer ring, then press the **OK button**.

The year is **blinking**.

- To change the year, press the **▲** or **▼ button**, then press the **OK button**.
- To change the month, press the **▲** or **▼ button**, then press the **OK button**.
- To change the day, press the **▲** or **▼ button**, then press the **OK button**.

**Setup is Complete**

The meter briefly displays your saved settings, then beeps and turns **OFF**.

**Setup is complete.**

**Fingertip Testing**

Refer to your lancing device insert for detailed instructions on preparing the lancing device.

**WARNING: Potential Biohazard**

- Always wash your hands well with soap and water and dry them before and after testing or after handling the meter, lancing device, or test strips.
- All parts of this kit are considered biohazardous and can potentially transmit infectious diseases.

For more information, refer to the CONTOUR NEXT ONE User Guide.

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**Need Help?**

For more information, go to [www.contourone.com](http://www.contourone.com)  
Contact Diabetes Care Customer Service at 1-800-268-7200.

Your meter is preset and locked to display results in mmol/L. If your result is in mg/dL, contact Diabetes Care Customer Service at 1-800-268-7200.

Manufactured for:  
Ascensia Diabetes Care Holdings AG  
Peter Merian-Strasse 90  
4052 Basel, Switzerland

**ASCENSIA**  
Diabetes Care

For in vitro diagnostic use.  
For information on patents and related licenses, see [www.patents.ascensia.com](http://www.patents.ascensia.com).

Ascensia, the Ascensia Diabetes Care logo, Contour, Glucofacts, the No Coding logo, Microlet, and Second-Chance are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. TM see [www.ascensiadabetes.ca/en/tm-mc](http://www.ascensiadabetes.ca/en/tm-mc)

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

The **Bluetooth**® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Ascensia Diabetes Care is under license. Google Play and the Google Play logo are trademarks of Google Inc. (® = registered trademark, ™ = unregistered trademark, SM = service mark.)

All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.

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Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

The **Bluetooth**® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Ascensia Diabetes Care is under license. Google Play and the Google Play logo are trademarks of Google Inc. (® = marque déposée, ™MC = marque commerciale non déposée, MS = marque de service.)

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## Fingertip Testing (Continued)

### CAUTION

For instructions on removing the lancet from your lancing device, see the lancing device insert.

The meter displays your result with the units, time, date, and target indicator: In Target , Below Target , or Above Target .

The test strip port displays a colour representing your result value compared to the Target Range.

In Target Below Target Above Target

## LO or HI Results

- If the meter beeps twice and displays the **LO** screen, your blood sugar reading is under 0.6 mmol/L. **Follow medical advice immediately.** Contact your health care professional.

- If the meter beeps once and displays the **HI** screen, your blood sugar reading is over 33.3 mmol/L.

- Wash and dry your hands well.
- Retest with a new strip.

If results are still over 33.3 mmol/L, **follow medical advice immediately.**

To turn the meter off, remove the test strip.

## Logbook

- To turn on your meter, press and hold the **OK button** for 2 seconds.

The **Home** screen has 2 options: **Logbook** and **Settings**.

The **Logbook**  is **blinking**.

- To view your **Logbook** entries, press the **OK button**.
- Press the **▲** and **▼ buttons** on the outer ring to scroll through the **Logbook**.

## Settings

You can turn settings such as **Sound** and **Target Light (smartLIGHT)** On and Off in **Settings**.

- To highlight the **Settings** symbol  on the **Home** screen, press the **▼ button** on the outer ring.
- When the **Settings** symbol is **blinking**, press the **OK button** to enter **Settings**.

## Turn Meal Markers On

If your meter is not **ON**:

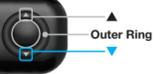
- Press and hold the **OK button** about 2 seconds until the meter turns on.

- Wait until the **Logbook** symbol is **blinking** on the **Home** screen.

- Press the **▼ button** on the outer ring.

- When the **Settings** symbol is **blinking**, press the **OK button**.

- To accept the change, press the **OK button**.

To exit from the **Logbook** or the last **Settings** screen and return to **Home**, press the **OK button**.

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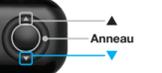
## Select a Meal Marker During a Blood Sugar Test

### Meal Marker Symbols

**NOTE:** You must do a blood sugar test before you can select a **Meal Marker**. You cannot select a **Meal Marker** in **Settings**.

- Test your blood sugar. See *Fingertip Testing* in this guide.
- Do not remove the test strip until you select a **Meal Marker**.

A **Meal Marker** blinks on the blood sugar result screen:

- If the **blinking Meal Marker** is the one you want, press the **OK button**.

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To select a different **Meal Marker**, press the **▲** or **▼ button** on the outer ring to scroll between markers.

- When the **Meal Marker** you want is **blinking**, press the **OK button**.

### NOTE:

- If you do not select a **Meal Marker** within 30 seconds, the screen dims. Press any button to turn the screen back on and make a selection.
- If you do not select a **Meal Marker** within 3 minutes, the meter turns off. Your blood sugar reading is stored in the **Logbook** without a **Meal Marker**.

## Your Meter Display

Symbol	What it Means
	Your Logbook
	Settings
	Target Light (smartLIGHT) Setting (Green=In Target; Red=Below Target; Yellow=Above Target)
	A Target range or Target setting is displaying
	Fasting Meal marker; used when testing after fasting (no food or drink for 8 hours, except water or non-caloric beverages)
	Before Meal marker; used when testing within 1 hour before a meal
	After Meal marker; used when testing within 2 hours after the first bite of a meal

Symbol	What it Means
	No Mark Selected; used when testing at times other than after fasting or before or after a meal
	Blood sugar test result is in Target range
	Blood sugar test result is above Target range
	Blood sugar test result is below Target range
	<b>Bluetooth</b> symbol; indicates the <b>Bluetooth</b> setting is On; the meter can communicate with a mobile device
	Batteries are low and need to be changed
	Indicates a meter error

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